

AED INSTRUCTIONS

1 Establish Unresponsiveness

Tap the victim and shout, "Are you OK?"

2 Call 9-1-1 and get the AED

If another person is present—have one person call 9-1-1 and the other get the AED.

3 Open Airway, Check Pulse


If no pulse, turn on the AED, begin CPR and follow AED voice prompts.

4 Prepare Victim for AED Use

Remove all clothing from victim's chest. Clean and dry skin, if needed.


5 Remove Seal from Pads

Have one person continue CPR while the other removes the seal from pads.

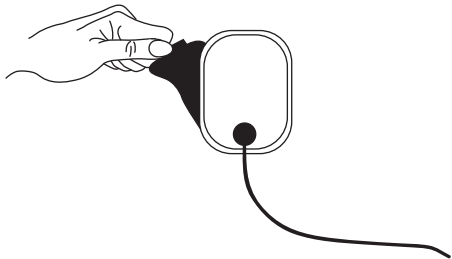

6 Place Pads on Victim

Place pads exactly as shown.

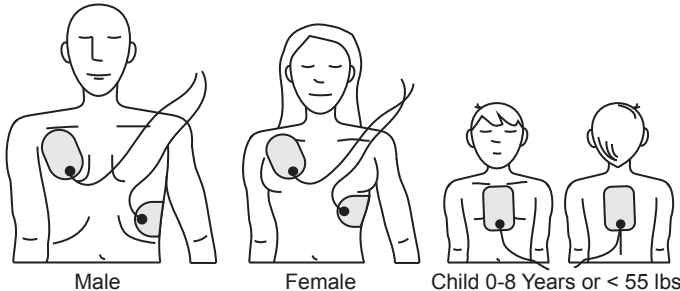

7 Prepare for Shock

The AED will analyze if shock is needed. If shock is advised, yell "**CLEAR!**" Make sure **no one is touching the victim**. Press the "Shock" button.

8 Restart Chest Compressions

Restart chest compressions and follow AED voice prompts. AED will analyze if another shock is needed.

9 Stop Compressions if Breathing Resumes

If patient begins breathing normally, stop compressions and place patient on their left side. Keep the AED attached until EMS arrive.

